



umano medical

QUICK REFERENCE CARD

Pilot Drive™

1 To activate the PILOT DRIVE



Unplug the bed.



Raise both handles at head end of the bed and insert the handles into the handle sockets.



Verify the battery charge level of the PILOT DRIVE before moving the bed.



Engage the steer position on the pedal.

The pedal can be engaged into steer position at any time in the process, but for security reasons, we suggest that this step be done when you are ready to move the bed.



The fifth wheel will descend in preparation for the Pilot Drive to be operated.

2 To use the PILOT DRIVE

Forward Function



To move the bed forward, press the trigger on the right handle.

To stop the bed, release the trigger.

Backwards Function



To move the bed backwards, press and hold the **R** button found on the top right handle, and press the trigger.

To stop the bed, release the **R** button or the trigger.

The more pressure applied on the trigger, the faster the bed will move.

3 To deactivate the PILOT DRIVE



Pull the handles out of the sockets and fold them down






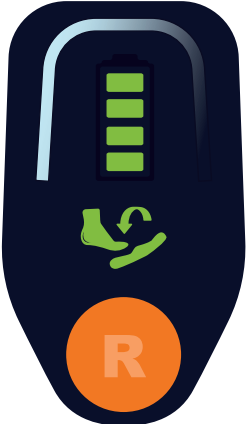

Apply the brakes.


For immediate assistance, please contact your local sales representative.

! Bed equipped with Pilot Drive option should only be operated by trained personnel. Please refer to the user manual for detailed instructions, safety measures, and warnings.



PILOT DRIVE Handle Indicators

	 BATTERY INDICATOR	 MOTION INDICATOR	 DRIVE INDICATOR
	<p>When the bed is plugged in</p> <ul style="list-style-type: none"> The 3 lower battery LED sections are green and the 4th battery LED section flashes green indicates that the battery is fully charged. The 3 lower battery LED sections flash green indicates that the battery is charging. <p>When the bed is unplugged</p> <ul style="list-style-type: none"> The number of battery LED sections that light up green indicates the level of battery charge remaining. 1 battery LED section is red indicates that the battery level is low.  	<ul style="list-style-type: none"> Motion LED outline gradually fills up white indicates that the bed is moving forward. <i>The motion indicator will fill up as more pressure is applied to the trigger</i> Motion LED lights up orange indicates that the bed is moving backward. If the battery LED flashes green when attempting to move the bed, this indicates that that the power cord needs to be unplugged. 	<ul style="list-style-type: none"> Drive LED is green indicates that the system is ready to be used. Drive LED flashes green indicates that the bed brakes are still engaged and must be placed in steer position.

 Bed equipped with Pilot Drive option should only be operated by trained personnel. Please refer to the user manual for detailed instructions, safety measures, and warnings.